- I** Foglio istruzioni - **Analogo laboratorio**
- GB** Information leaflet - **Implant analog**
- D** Gebrauchsanweisung - **Labor analoge**
- E** Hoja de instrucciones - **Análogo laboratorio**
- CZ** Návod k použití - **Analog (laboratorní implantát)**

RIUTILIZZABILE
 Riutilizzabile
 Reusable
 Wiederverwendbar
 Reutilizable
 Možnost opakovaného použití

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GB

GENERAL WARNING

- Dental Tech implant analogs shall be used by specialised medical staff and dental technicians only.
- The use of prosthesis components not belonging to the endosseous system manufactured by Dental Tech, may impair its functionality.
- The product is manufactured and applied out of our control and responsibility. The manufacturing company cannot be held responsible for any deriving damage.
- Prior to using any Dental Tech product inspect the packaging and device for damages.
- At the end of its useful life the medical device must be disposed of in accordance with national laws regarding waste disposal.
- Dental Tech cannot be held responsible for any damage resulting from inobservance of this warning.

INDICATION

- Dental Tech implant analogs are used in laboratory when manufacturing master samples aimed at faithfully reproducing the position of implants as in the oral cavity.
- Prior to manufacturing the working sample, connect the respective implant impressions by the kind of implant or prosthetic part used in the oral cavity.
- Dental Tech implant analogs for abutment E.M. shall be used exclusively matching the abutment E.M.
- Dental Tech implant analogs for abutment E.M. are used in a laboratory when manufacturing master samples aimed at faithfully reproducing the position of abutments E.M. as in the oral cavity.

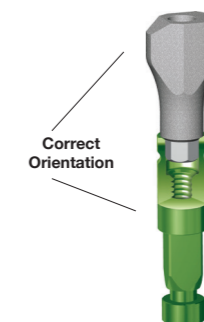
CONTRAINDICATIONS

- Use it for the above-mentioned purpose only.
- Do not change the original structure of analogs.

DIGITAL ANALOGUE

- Analogue for digital models, specifically for applications using models manufactured with 3D printing/prototyping techniques. The typical shape with radial corners allows easy insertion into the site of the model produced, without either interferences or friction with the resinous materials of the models. The apical screw always ensures absolute functional stability. This prosthetic component must be used with Dental Tech Digital Libraries.

DIGITAL ANALOGUE - INDICATION FOR USE

WARNING: DO NOT direct the Scan Abutment in other unsuitable positions.

The smaller portion of the Scan Abutment, which is aligned with the hexagon side of the connection, **must always** match the milled site located on the cylindrical portion of the digital analogue body.

PACKAGING

- Dental Tech tools are supplied as non sterile in heat-sealed pouches unsuitable for autoclave sterilisation.

CLEANING, DISINFECTION AND STERILIZATION

- Dental Tech tools can be used again and therefore it has to be washed, disinfected and sterilised prior to their usage.
- Cleaning, disinfection and sterilization operations must be carried out by qualified personnel under the direct responsibility of the doctor; sterilization must be carried out through the use of regularly calibrated machinery and in perfect state.
- Wear disposable gloves during all cleaning, disinfection and sterilization operations.

Dental Tech has validated the manual cleaning and disinfection method below. Do not use automated cleaning and disinfection methods because they are not validated.

PREPARATION TO THE POINT OF USE

- Set aside damaged or beveled tools.
- Proceed with manual cleaning without letting the organic material dry on the instruments.

MANUAL CLEANING

- Just after the use of Dental Tech equipment, place the equipment into ultrasound bath with a peracetic acid based solution at concentration of 2% (NO GLUTAR- ALDEHYDE OR SODIUM HYPOCHLORITE), as long as 18 minutes.
- Afterward rinse carefully with drinking water.

MANUAL DISINFECTION

- Place the equipment into a container with a peracetic acid based solution at concentration of 4% (NO GLUTARALDEHYDE OR SODIUM HYPOCHLORITE), as long as 15 minutes.
- Rinse generously with drinking water.
- Examine the equipment and make sure there are no organic remains. Carefully scrub the outer parts with a non-metal rigid bristled brush.

MANUAL RINSE

- Place the equipment into ultrasound bath with distilled water, and wash it for approx. 18 minute and then rinse carefully with distilled water.

- DRY and BAG

• Dry perfectly in aseptic conditions (gauze, paper, filtered air...) and store the instruments individually bagged using Medical Paper C and FILM PET/PP Peel (ISO Reference 11607-1).

• STERILIZATION (according to ISO 17665-1)

• Dental Tech validated the following Autoclave moist heat sterilization cycle:

• Pre-vacuum time according to autoclave builder specifications.

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