- I** Foglio istruzioni - **Basi per incollaggio**
- GB** Information leaflet - **Bonding bases**
- D** Gebrauchsanweisung - **Halteelemente für Klebetechnik**
- E** Hoja de instrucciones - **Bases para pegado**
- CZ** Návod k použití - **Báze pro lepení**

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GB

GENERAL WARNING

- The Dental Tech bases for bonding must only be used by specialized medical personnel and orthodontic technicians
- The use of prosthetic components not belonging to the endosseous system produced by Dental Tech, may compromise functionality.
- The processing and application of the product, occur outside of our control and of our responsibility. Dental Tech assumes no responsibility in the event of injury incurred from their application.
- Check the integrity of the appliance before use.
- At the end of its useful life, the medical appliance must be disposed of in compliance with the manner established by the national laws in force relating to waste.
- Re-use of the appliance is prohibited. If the technician decides to violate this prohibition, it may expose the patient to risks of infection, transmittable disease through contact with blood or body fluids of the patient, deformation of the appliance and resulting impossibility of coupling with associated medical devices (implants, instruments, etc.)
- Dental Tech declines any responsibility in the event of failure to respect the regulations listed above.

INDICATION

- All Dental Tech bases for cementing are created with a thickness reduced to a minimum to limit encumbrance; further grinding is not advised.
- The bases may be shortened in the case of reduced occlusal space; for grinding it is advised to use burrs specific for titanium. If modified/corrected, the user must check whether they can still be used with CAD-CAM techniques.
- After having created the prosthetic, it is advisable, before temporary and / or final cementing, to protect the fastening screw with appropriate material.
- Pre-conditioning of the soft tissue is necessary using the dedicated gum conditioners.
- Tighten the fastening screw of the bases with the 20 N cm Dental Tech torque wrench and the appropriate adapter for the torque wrench, cod. TW0001 C – TW0001 L.
- Tighten the BCM bases with GMM 250 and 001152 manual screwdrivers.
- Do not solder or braze.
- The Dental Tech bases for bonding are divided into the following categories:

- **BONDING BASES ENGAGING** are indicated for the technique of bonding and may be processed using a digital CAD-CAM system and with the traditional technique. They are used for the creation of single cemented and screwed elements for the creation of cemented bridges. Being titanium bases with hexagon anti-rotational, they are not advised for the creation of screwed bridges. In case of single elements, they serve as the connection base with the implant on which individual abutments and /or single crowns, created with aesthetic materials such as Zirconium and Lithium Disilicate, are bonded. In the case of cemented bridges, they serve as the base of connection

with the implant on which individual abutments, created with aesthetic materials such as Zirconium and Lithium Disilicate, are bonded.

- **BONDING BASES NON-ENGAGING** are indicated for the technique of bonding and may be processed using a digital CAD-CAM system and with the traditional technique. They are used for the creation of multiple elements screwed on implants when there are unilateral edentulous and full arch spaces, where they serve as the connection base with implants in the case of an Implant bridge and / or Toronto Bridge on which it is possible to bond metallic structures and / or Zirconium Oxide structures. In case of titanium bases without an anti-rotational hexagon, their use is not advised for the creation of single cemented and screwed elements for the creation of cemented bridges.
- **BONDING BASES FOR MUA** are indicated for the technique of cementing and may be processed using a digital CAD-CAM system and with the traditional technique. They are used for the creation of multiple elements screwed on MUA abutments where there are unilateral edentulous and full arch spaces; they are connected to MUA abutments in case of an Implant bridge and /or Toronto bridge on which it is possible to bond metallic structures and / or Zirconium Oxide structures. In the case of titanium bases without an anti-rotational hexagon, their use is not advised for the creation of single cemented and screwed elements for the creation of cemented bridges. For the cementing of bases to the superstructures commercialized adhesive cements are used, such as those of 3M Espe and Bredent.
- **BONDING BASES FOR ANGLED SCREW CHANNEL:** They can be used with traditional or CAD-CAM techniques. The portion dedicated to receiving the aesthetic bonded prosthetic reconstruction can be modified/corrected. If used with the CAD-CAM technique, choose the most suitable .stl file from the digital libraries and check the correspondence of the modifications implemented.

CONTRAINDICATIONS

- Do not tighten or loosen the fastening screws using common tools.
- Do not use for purposes different to those indicated and with a divergence greater than 10° for straight pillars.

PACKAGING AND STERILITY

- The appliance is single use.
- "Sterilize in a steam autoclave with distilled water using a validated and systematically controlled method of sterilization in accordance with the requirements of ISO Standard 17665-1:2007 "Sterilization of health care products". An example of a cycle could be the sterilization, using an autoclave able to eliminate the air pockets which form within the sterilization chamber, for 20 minutes at a temperature of 134°C and a pressure of 2 Atm".

WARNING

Pouches unsuitable for autoclave sterilisation.
N.B. Cleaning and sterilisation shall be carried out by qualified personnel, under the supervision of a doctor and by using properly calibrated and faultless machines.

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