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GB

GENERAL WARNING

- Dental Tech drills and instruments for guided surgery must only be used by specialised medical personnel who are thoroughly familiar with the guided surgery technique.
- The use of drills not belonging to the endosseous system manufactured by Dental Tech, may impair its functionality.
- The product is manufactured and applied out of our control and responsibility. The manufacturing company cannot be held responsible for any deriving damage. Do not use in case of damaged or open packaging.
- Prior to using any Dental Tech product inspect the packaging and device for damages.
- For functional and manufacturing reasons, all drills have a higher apical length. On the working portion of cylindrical drills depth notches are laser marked which allow a clear view even during rotation. Intermediate and conical drills are provided with a revolving and removable depth stop. Something very important to bear in mind for all drills is that the site preparation does not correspond to the implant length but rather to a higher depth ranging from 0.8 mm to max 1.0 mm (final drills). Therefore we recommend reaching a depth of 0.8-1 mm lower than the project.
- Drills are cutting tools. Take precautions when washing and cleansing them especially if they have been contaminated by recent use. Wear protection goggles, surgical mask and disposable gloves.
- At the end of its useful life the medical device must be disposed of in accordance with national laws regarding waste disposal.

- Dental Tech cannot be held responsible for any damage resulting from inobservance of this warning

INDICATION

- **DRILLS FOR GUIDED SURGERY:** in order to create the surgical site for the insertion of dental implants, we indicate Dental Tech drills for guided surgery as suitable instruments to mill the bone. They can also be used through a sleeve inserted in the surgical template previously designed by means of dedicated software for the creation of the specific receptor site. Drills for guided surgery feature a dedicated diameter and length laser-marked on the drill shank so that, based on the project performed, specialised medical personnel can assess the most suitable sequence of use for the clinical case to be treated.
- **SCREWDRIVERS FOR GUIDED SURGERY:** with or without a through screw, they are grouped by their dedicated prosthetic platform. Their correct use is specified in the corresponding section of the reference catalogue. The correct use of the screw extractor is also indicated in the corresponding section.
- **COLOURED SPACER RINGS (H 1 - 1.5 - 2 mm):** they must only be used by specialised medical personnel who are thoroughly familiar with the guided surgery technique. For guided surgery, they can be used with burs in order to reduce the cutting capacity of the drills, and with screwdrivers that lack a through screw for subcrestal insertion of the implant fixtures. Their use shall be established by specialised medical personnel based on the defined evaluation criteria.
- **Target population:** all patients eligible for guided surgery involving implants-prostheses.

METHOD OF USE

- Sprinkle drills generously and constantly with sterile physiological solution previously cooled down.
- Drills must be used together with a surgical micro-motor and constant saline irrigation by its special knob.
- Do not set any power higher than 55N/cm with micro-motors provided with TORQUE safety device when preparing the site through drills.

CONTRAINDICATIONS

- When using Dental Tech drills never lever or bend them.
- Use them for the above-mentioned purpose only.
- Do not use drills with rotation speed higher than 800 revolutions/minute.
- Do not wash drills and other instruments in the same bath, but carefully separate.
- To prevent tools from deteriorating and bones from overheating, do not use drills for more than 20 repetitions.
- Do not change its original structure.

PACKAGING

- Dental Tech tools are supplied as non sterile in heat-sealed pouches unsuitable for autoclave sterilisation.

CLEANING, DISINFECTION AND STERILIZATION

- Dental Tech tools can be used again and therefore it has to be washed, disinfected and sterilised prior to their usage.
- Cleaning, disinfection and sterilization operations must be carried out by qualified personnel under the direct responsibility of the doctor;

sterilization must be carried out through the use of regularly calibrated machinery and in perfect state.

- Wear disposable gloves during all cleaning, disinfection and sterilization operations.

Dental Tech has validated the manual cleaning and disinfection method below. Do not use automated cleaning and disinfection methods because they are not validated.

PREPARATION TO THE POINT OF USE

- Disassemble the drills from the handpiece immediately after use

- Set aside damaged or beveled tools.

- Proceed with manual cleaning without letting the organic material dry on the instruments.

MANUAL CLEANING

- Just after the use of Dental Tech equipment, place the equipment into ultrasound bath with a peracetic acid based solution at concentration of 2% (NO GLUTAR- ALDEHYDE OR SODIUM HYPOCHLORITE), as long as 18 minutes.

- Afterward rinse carefully with drinking water.

MANUAL DISINFECTION

- Place the equipment into a container with a peracetic acid based solution at concentration of 4% (NO GLUTARALDEHYDE OR SODIUM HYPOCHLORITE), as long as 15 minutes.

- Rinse generously with drinking water.

- Examine the equipment and make sure there are no organic remains. Carefully scrub the outer parts with a non-metal rigid bristled brush.

MANUAL RINSE

- Place the equipment into ultrasound bath with distilled water, and wash it for approx. 18 minute and then rinse carefully with distilled water.

- DRY and BAG

- Dry perfectly in aseptic conditions (gauze, paper, filtered air..) and store the instruments individually bagged using Medical Paper C and FILM PET/PP Peel (ISO Reference 11607-1).

- STERILIZATION (according to ISO 17665-1)

- Dental Tech validated the following Autoclave moist heat sterilization cycle:

- Pre-vacuum time according to autoclave builder specifications.

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