Italiano

I

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GB

GENERAL WARNING

- Dental Tech equipment shall be used by specialised medical staff and dental technicians only.
- The use of equipment not belonging to the endosseous system manufactured by Dental Tech, may impair its functionality.
- The product is manufactured and applied out of our control and responsibility. The manufacturing company cannot be held responsible for any deriving damage.
- Prior to using any Dental Tech product inspect the packaging and device for damages.
- At the end of its useful life the medical device must be disposed of in accordance with national laws regarding waste disposal.
- Dental Tech cannot be held responsible for any damage resulting from inobservance of this warning.

CONTRAINDICATIONS

- Use them for the above - mentioned purpose only.
- Do not modify the original structure of the equipment.
- Do not wash the equipment with different material in the same bath, but carefully separated.

PACKAGING

- Dental Tech tools are supplied as non sterile in heat-sealed pouches in containing the leaflet.
- Pouches unsuitable for autoclave sterilisation.

CLEANING, DISINFECTION AND STERILIZATION

- Dental Tech tools can be used again and therefore it has to be washed, disinfected and sterilised prior to their usage.
- Cleaning, disinfection and sterilization operations must be carried out by qualified personnel under the direct responsibility of the doctor; sterilization must be carried out through the use of regularly calibrated machinery and in perfect state.
- Wear disposable gloves during all cleaning, disinfection and sterilization operations.

Dental Tech has validated the manual cleaning and disinfection method below. **Do not use automated cleaning and disinfection methods because they are not validated.**

PREPARATION TO THE POINT OF USE

- Set aside damaged or beveled tools.
- Proceed with manual cleaning without letting the organic material dry on the instruments.

MANUAL CLEANING

- Just after the use of Dental Tech equipment, place the equipment into ultrasound bath with a peracetic acid based solution at concentration of 2% (NO GLUTAR- ALDEHYDE OR SODIUM HYPOCHLORITE), as long as 18 minutes.
- Afterward rinse carefully with drinking water.

MANUAL DISINFECTION

- Place the equipment into a container with a peracetic acid based solution at concentration of 4% (NO GLUTARALDEHYDE OR SODIUM HYPOCHLORITE), as long as 15 minutes.
- Rinse generously with drinking water.
- Examine the equipment and make sure there are no organic remains. Carefully scrub the outer parts with a non-metal rigid bristled brush.

MANUAL RINSE

- Place the equipment into ultrasound bath with distilled water, and wash it for approx. 18 minute and then rinse carefully with distilled water.

DRY and BAG

- Dry perfectly in aseptic conditions (gauze, paper, filtered air...) and store the instruments individually bagged using Medical Paper C and FILM PET/PP Peel (ISO Reference 11607-1).

STERILIZATION (according to ISO 17665-1)

Dental Tech validated the following Autoclave moist heat sterilization cycle:

- Pre-vacuum time according to autoclave builder specifications.
- Temperature: 134 °C.
- Holding time: 3 minutes.
- Drying time: minimum 10 minutes.

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