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English**GB****GENERAL WARNING**

- Dental Tech plastic sleeves shall be used by specialised medical staff and dental technicians only.
- The use of prosthesis components not belonging to the endosseous system manufactured by Dental Tech, may impair its functionality.
- The product is manufactured and applied out of our control and responsibility. The manufacturing company cannot be held responsible for any deriving damage.
- Prior to using any Dental Tech product inspect the packaging and device for damages.
- At the end of its useful life the medical device must be disposed of in accordance with national laws regarding waste disposal.
- Reuse of the device is prohibited. If the operator decides to violate this prohibition, the patient may be exposed to risks of infections, diseases transmitted through contact with blood or bodily fluids, and deformation of the device, make it impossible to match with related medical devices (implants, instruments, etc.)
- Dental Tech cannot be held responsible for any damage resulting from inobservance of this warning.

I Foglio istruzioni - Pilastro calcinabile**GB Information leaflet - Plastic sleeve****D Gebrauchsanweisung - Angussfähiger kunststoff zylinder****E Hoja de instrucciones - Pilar calcinable****CZ Návod k použití - Spalitelný abutment****DIRECTIONS**

- Dental Tech plastic sleeve with abutment screw is suitable for assembling Ackerman, Dolder removable abutment bars including those deriving from electro-erosion. It is suitable for cementum-covered prosthesis.
- Accomplish the melting operation on Dental Tech plastic sleeves by using either noble or titanium alloy when, coming into contact with the patient's soft tissues, it has to have biocompatibility features.
- On melting, remember to keep the final temperature approx. 15 minutes longer to allow the full combustion of the sleeve material in the casting chamber.
- The abutment screw for plastic sleeve code **112218** and code **112219** can be reduced in its connecting part with the tool to screw it up to the line marking its maximum adjustment limit.
- Tighten the abutment screw through Dental Tech dynamometrical wrench at 20 N cm and the special adapter for dynamometrical wrench code **TW0001 C – TW0001 L**.
- 112216/112250** plastic sleeves shall be used exclusively matching the octagonal plastic sleeve code **112215**.
- In the event of more implants, revolving plastic sleeves code **112250** can ease the insertion of bars or permanent elements, thus smoothing the problem of disparallelism.

CONTRAINDICATIONS

- Do not screw or unscrew the abutment screw for plastic sleeve with unsuitable tools.
- Do not use melting alloys which may impair its biocompatibility.
- Dental Tech plastic sleeve with abutment screw is not suitable for making immediate temporary devices.
- Dental Tech plastic sleeve with abutment screw code **003324/12216/HPE4160** is not suitable for making single implant-prosthetic elements with melting alloys that are less than 200 vickers in hardness.
- Do not use plastic sleeves in the patient's oral cavity before it has been shaped and changed.
- CCEM / HMC4100** plastic sleeves shall not be used for cementum-covered prostheses and for single implant-prosthetic elements.
- Revolving plastic sleeves **code112250/112218/CCEM/HPR0041/CBR3200/PCR037** shall not be used for single implant-prosthetic elements.

PACKAGING AND STERILITY

- The device is disposable.
- After being transformed, the device must be cleaned and washed. The device does not need to be sterilised after transformation.

WARNING**Pouches unsuitable for autoclave sterilisation.**

N.B. Cleaning and sterilisation shall be carried out by qualified personnel, under the supervision of a doctor and by using properly calibrated and faultless machines.

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