Italiano

I

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GB

GENERAL WARNING

- Dental Tech ball retainers shall be used by specialised medical staff and dental technicians only.
- The use of prosthesis components not belonging to the endosseous system manufactured by Dental Tech, may impair its functionality.
- The product is manufactured and applied out of our control and responsibility. The manufacturing company cannot be held responsible for any deriving damage.
- Prior to using any Dental Tech product inspect packaging and device for damages.
- At the end of its useful life the medical device must be disposed of in accordance with national laws regarding waste disposal.
- Reuse of the device is prohibited. If the operator decides to violate this prohibition, the patient may be exposed to risks of infections, diseases transmitted through contact with blood or bodily fluids, and deformation of the device, make it impossible to match with related medical devices (implants, instruments, etc.)

I Foglio istruzioni - **Attacco sfera**
GB Information leaflet - **Ball retainer**
D Gebrauchsanweisung - **Kugelkopfanker**
E Hoja de instrucciones - **Pilar de bola**
CZ Návod k použití - **Attachment kulový**

- Dental Tech cannot be held responsible for any damage resulting from inobservance of this warning.

INDICATIONS

- Dental Tech mounted attachments are the most interesting solution for immediate anchoring of tooth replacement implants.
- The mounted attachment are indicated as a temporary base while waiting for the other implants to integrate with the bone
- The Dental Tech mounted attachments can be made into prostheses with a system composed of a titanium retention cylinder code POR225 and silicone o-ring ORG225, or alternatively a specific crown and related steel containers
- It is advisable to replace the o-ring following assessment of the hold of the prosthesis. If using crowns follow the instructions of the manufacturer.
- Use the specific analog, to take an impression of the Dental Tech mounted attachment.
- Tighten the mounted attachment with a Dental tech torque wrench to 30 N cm and the specific adapter for torque wrench code RDS225.

CONTRAINDICATIONS

- Do not use Dental Tech ball retainers for implants diverging more than 10°.
- Do not change the original structure of Dental Tech ball retainers.
- Do not use unsuitable screw-drivers (universal pliers etc.).
- They are not recommended when there are less than two implants.
- Do not cover the implant with cementum.

PACKAGING AND STERILITY

- Dental Tech ball retainers are supplied as non sterile in heat-sealed Pouches in cardboard boxes containing the leaflet.
- The device is disposable.
- “Autoclave moist heat sterilisation must be carried out by using an validated and routinely checked method pursuant the requirements as per ISO 17665-1:2007 *“Sterilization of health care products”*. An example of cycle may be represented by autoclave sterilisation able to get rid of any air pocket within the sterilisation chamber for 20 minutes at a temperature equal to 134°C and pressure equal to 2 atm”.

WARNING

Pouches unsuitable for autoclave sterilisation.

N.B. Cleaning and sterilisation shall be carried out by qualified personnel, under the supervision of a doctor and by using properly calibrated and faultless machines.

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