Italiano I

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English GB

GENERAL WARNING

- Dental Tech abutments shall be used by specialised medical staff and dental technicians only.
- The use of prosthesis components not belonging to the endosseous system manufactured by Dental Tech, may impair its functionality.
- The product is manufactured and applied out of our control and responsibility. The manufacturing company cannot be held responsible for any deriving damage.
- Prior to using any Dental Tech product inspect the packaging and device for damages.
- At the end of its useful life the medical device must be disposed of in accordance with national laws regarding waste disposal.
- Reuse of the device is prohibited. If the operator decides to violate this prohibition, the patient may be exposed to risks of infections, diseases transmitted through contact with blood or bodily fluids, and deformation of the device, make it impossible to match with related medical devices (implants, instruments, etc.)
- Dental Tech cannot be held responsible for any damage resulting from inobservance of this warning.

DIRECTIONS

- Dental Tech titanium base with plastic sleeve is recommended for permanent and temporary prostheses (overdenture). You can make screwed and/or cementum-covered structures, bridges, single crowns, bars.
- The coronal portion of the Dental Tech plastic sleeve with titanium base shall be changed by the technician through waxing and lost wax casting.
- The coronal portion of the Dental Tech plastic sleeve with titanium base changed or cast by the technician shall be connected to the titanium transmucosal portion through the special abutment screw (by soldering it if casting is performed in titanium).
- Straight/pre-angled abutments are suitable for permanent cementum-covered prostheses, prep abutments for cementum-covered and taper prostheses.
- Straight/pre-angled abutments can be adjusted and, thanks to its anti-rotational system you can use the indirect technique by using impressions keeping the surgical position of the implant in the impression, and analogs reflecting the inside part and the neck of the implant.
- To adjust abutments we recommend using drills specifically designed for titanium.
- Pre-angled abutments are supplied with pre-cast angles (see chart), therefore very useful to solve any disparallelism up to 25° either with more implants, or natural toothings.
- Screwable straight abutments are suitable for permanent prostheses.
- Straight/prep abutments can be adjusted to reach up to a minimum thickness of 0.5 mm in the coronal portion and 0.6 mm in the cervical portion.

- After manufacturing the prosthesis, before permanent and/or temporary casting, we suggest protecting the abutment screw with special material.
- For straight/pre-angled abutments we suggest using a thread dental sealant to increase the grip of screwable abutments.
- Tighten straight screwable abutments through Dental Tech torque wrench at 20 N cm and the special adapter for torque wrench.
- Tighten straight screwable abutments through Dental Tech torque wrench at 30 N cm and the special adapter for torque wrench.
- Soft tissues needs pre-conditioning through special healing abutments.
- Tighten abutment screws with Dental Tech torque wrench at 20 N cm and the special adapter for torque wrench.
- Do not weld or solder.
- Do not use screwable abutments in the event of single elements.
- If the transmucosal part is to be adjusted, do not exceed the connecting area to the implant.

N.B. For information on codes, measurements and characteristics of individual articles, consult the "Dental Implant System" catalog on our website www.dentaltechitalia.com.

CONTRAINDICATIONS

- Dental Tech plastic sleeve titanium base shall not be used in the event of screwed prosthesis (low bridges), if disparallelism among more implants and natural elements exceeds 15°.
- Do not screw or unscrew the abutment screw with unsuitable tools.
- Use for the above-mentioned purpose only and do not use with straight abutments diverging more than 10°.
- Use pre-angled abutments for the above-mentioned purpose only and do not use with pre-angled abutments diverging more than 25°.
- Do not screw or unscrew straight screwable abutments with unsuitable tools.
- The titanium abutment code CTEM shall not be used for single implant-prosthetic elements.

PACKAGING AND STERILITY

- This device is disposable.
- "Autoclave moist heat sterilisation must be carried out by using an validated and routinely checked method pursuant the requirements as per ISO 17665-1:2007 "Sterilization of health care products". An example of cycle may be represented by autoclave sterilisation able to get rid of any air pocket within the sterilisation chamber for 20 minutes at a temperature equal to 134°C and pressure equal to 2 atm".

WARNING Pouches unsuitable for autoclave sterilisation.

N.B. Cleaning and sterilisation shall be carried out by qualified personnel, under the supervision of a doctor and by using properly calibrated and faultless machines.

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