- I** Avvitatore manuale e da contrangolo
- GB** Manual and counterangle-connection screw-driver
- D** Einbringhilfe zur manuellen und maschinell einbringung
- E** Prolongador y conexión para contrángulo
- CZ** Použití zašroubujte manuálně a s kolénkem

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**GB**

**Information leaflet manual and counterangle-connection screw-driver for ImpLassic e ImpLogic®**

**IMA.../HMD25..** It is used to insert implants into the surgical sites together with torque and/or manual wrench.

**ICO.../HHP25..** It is used to mechanically insert the implants into the surgical site connected to the counterangle.

*The laser-marked notches on the device help the operator place the dental implant in the correct position when FLAP-LESS surgery is performed.*

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## English

### GENERAL WARNING

- Dental Tech equipment shall be used by specialised medical staff and dental technicians only.

- The use of equipment not belonging to the endosseous system manufactured by Dental Tech, may impair its functionality.

- The product is manufactured and applied out of our control and responsibility.

- The manufacturing company cannot be held responsible for any deriving damage.

- Prior to using any Dental Tech product inspect the packaging and device for damages.

- At the end of its useful life the medical device must be disposed of in accordance with national laws regarding waste disposal.

- Dental Tech cannot be held responsible for any damage resulting from inobservance of this warning.

### CONTRAINDICATIONS

- Use them for the above -mentioned purpose only.

- Do not modify the original structure of the equipment.

- Do not wash the equipment with different material in the same bath, but carefully separated.

### PACKAGING

- Dental Tech tools are supplied as non sterile in heat-sealed pouches in containing the leaflet.

- Pouches unsuitable for autoclave sterilisation.

### CLEANING, DISINFECTION AND STERILIZATION

- Dental Tech tools can be used again and therefore it has to be washed, disinfected and sterilised prior to their usage.

- Cleaning, disinfection and sterilization operations must be carried out by qualified personnel under the direct responsibility of the doctor; sterilization must be carried out through the use of regularly calibrated machinery and in perfect state.

- Wear disposable gloves during all cleaning, disinfection and sterilization operations.

- Dental Tech has validated the manual cleaning and disinfection method below. Do not use automated cleaning and disinfection methods because they are not validated.

### PREPARATION TO THE POINT OF USE

- Set aside damaged or beveled tools.

- Proceed with manual cleaning without letting the organic material dry on the instruments.

### MANUAL CLEANING

- Just after the use of Dental Tech equipment, place the equipment into

- ultrasound bath with a peracetic acid based solution at concentration of 2% (NO GLUTARALDEHYDE OR SODIUM HYPOCHLORITE ), as long as 18 minutes.

- Afterward rinse carefully with drinking water.

### MANUAL DISINFECTION

- Place the equipment into a container with a peracetic acid based solution at concentration of 4% (NO GLUTARALDEHYDE OR SODIUM HYPOCHLORITE), as long as 15 minutes.

- Rinse generously with drinking water.

- Examine the equipment and make sure there are no organic remains. Carefully scrub the outer parts with a non-metal rigid bristled brush.

### MANUAL RINSE

- Place the equipment into ultrasound bath with distilled water, and wash it for approx. 18 minute and then rinse carefully with distilled water.

### DRY AND BAG

- Dry perfectly in aseptic conditions (gauze, paper, filtered air...) and store the instruments individually bagged using Medical Paper C and FILM PET/PP Peel (ISO Reference 11607-1).

**STERILIZATION (according to ISO 17665-1)**
Dental Tech validated the following Autoclave moist heat sterilization cycle:

- Pre-vacuum time according to autoclave builder specifications.

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