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English**GB****GENERAL WARNING**

- The Dental Tech Scan Abutments must only be used by specialized medical personnel and orthodontic technicians
- The use of prosthetic components not belonging to the endosseus system produced by Dental Tech, may compromise functionality.
- The processing and application of the product, occur outside of our control and of our responsibility. Dental Tech assumes no responsibility in the event of injury incurred from their application.
- Check the integrity of the appliance before use.
- At the end of its useful life, the medical appliance must be disposed of in compliance with the manner established by the national laws in force relating to waste.
- Dental Tech declines any responsibility in the event of failure to respect the regulations listed above.

INDICATIONS

Dental Tech SCAN ABUTMENTS are indicated for the taking of masks with digital techniques, may be used to perform a scan in the oral cavity of the patient using an intraoral scanner.

They may also be used on clay models, created with the standard impression technique, combined with implant analogs; in this case the scan may be performed with a laboratory scanner or with a multifunctional scanner (intraoral / laboratory).

The section of the appliance which is scanned is created in a plastic material (PEEK); therefore it is not necessary to matte them with powdered spray.

Dental Tech SCAN ABUTMENTS for single cemented and screwed elements, and for multiple cemented elements, are fastened to the implant with special fastening screws and special screwdrivers (cod. **001152** and **GMM 250**).

Dental Tech SCAN ABUTMENTS for multiple screwed elements for both implants and the EM pillar are fastened manually to the implant and / or pillar. THE SIRONA TYPE PILLAR CAP connects to the pillar in a single position using light manual pressure.

CONTRAINDICATIONS

- Do not tighten or loosen the fastening screws for DENTAL TECH SCAN ABUTMENTS using common tools.
- Do not use for purposes different to those indicated.
- Do not alter the original structure of either the SCAN ABUTMENTS or the relative fastening screw.
- Dental Tech SCAN ABUTMENTS are not suitable for permanency in the oral cavity.
- The SCAN ABUTMENTS may not be retouched.

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PACKAGING AND STERILITY

- Dental Tech tools are supplied as non sterile in heat -sealed Pouches in containing the leaflet.
- Dental Tech tools can be used again and therefore it has to be washed and sterilised prior to their usage.

Dental Tech validated the following cleansing and disinfection method:

MANUAL CLEANING

- Just after the use of Dental Tech equipment, place the equipment into ultrasound bath with a peracetic acid based solution at concentration of 2% (NO GLUTARALDEHYDE OR SODIUM HYPOCHLORITE), as long as 18 minutes.
- After-ward rins

MANUAL DISINFECTION

- Place the equipment into a container with a peracetic acid based solution at concentration of 4% (NO GLUTARALDEHYDE OR SODIUM HYPOCHLORITE), as long as 15 minutes.
- Rinse generously
- Examine the equipment and make sure there are no organic remains. Carefully scrub the outer parts with a non-metal bristled brush.

MANUAL RINSE

- Place the equipment into ultrasound bath with distilled water, and wash it for approx. 18 minute and then rinse carefully.

DRY

- Perfectly dry the equipment, seal it individually with material suitable for moist heat sterilisation.

STERILIZATION

- Dental Tech validated the following Autoclave moist heat sterilization cycle:
3 minutes - 134 °C

- Since Dental Tech tools are manufactured in different materials, they shall be washed and sterilized one by one

WARNING

Pouches unsuitable for autoclave sterilisation.

N.B. Cleaning and sterilisation shall be carried out by qualified personnel, under the supervision of a doctor and by using properly calibrated and faultless machines.

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