- I** Foglio istruzioni - **Transfer strappo/precisione/sfera**
- GB** Information leaflet - **Closed-tray/precision/ball impressions**
- D** Gebrauchsanweisung - **Abdruckpfosten**
- E** Hoja de instrucciones - **Transfer de impresión**
- CZ** Návod k použití - **Otiskovací pilíře**

RIUTILIZZABILE
Riutilizzabile
Reusable
Wiederverwendbar
Reutilizable
Možnost opakovaného použití

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GB

GENERAL WARNING

- Dental Tech impressions shall be used by specialised medical staff and dental technicians only.
- The use of prosthesis components not belonging to the endosseous system manufactured by Dental Tech, may impair its functionality.
- The product is manufactured and applied out of our control and responsibility. The manufacturing company cannot be held responsible for any deriving damage.
- Prior to using any Dental Tech product inspect the packaging and device for damages.
- At the end of its useful life the medical device must be disposed of in accordance with national laws regarding waste disposal.
- Dental Tech cannot be held responsible for any damage resulting from inobservance of this warning.

INDICATION

- Dental Tech impressions are recommended to make impressions when performing traditional and/or indirect and/or precision techniques.

- TRADITIONAL AND/OR INDIRECT TECHNIQUE:

- Anchor Dental Tech impression to the implant by the special abutment screw
- Graft ball impressions onto the balls with its respective diameter (MICRO/ NORMO).
- Make the impression by using a standard or single impression-holder with high-elastic-memory specific materials.
- Wait until the material sets and then remove the impression-holder from the oral cavity. Unscrew the screw and remove the impression from the implant. After the impression has been checked and washed, anchor it to the plaster analog by the same screw and then replace the impression-analog-abutment screw set, into the impression previously made. As it is a precision stage, we recommend checking the actual correctness of the impression before dismissing the patient.

- INDIRECT AND/OR PRECISION TECHNIQUE:

- Anchor Dental Tech implant impressions to the implant through the special abutment screw protruding from the impression and the special screw-driver
- Make the impression by using only a single impression-holder with a hold by the implant so that the abutment screw stands out.
- Use high-elastic-memory material.
- Once the abutment screw has hardened, unscrew it, remove the impression (at this stage the impression is steady), and after the latter

has been washed, disinfected and checked, anchor the analog through the abutment screw carefully, keeping the initial position during all screwing and tightening procedures. Carefully carry out this procedure in order to allow the analog to be exactly anchored to the impression.

CONTRAINDICATIONS

- Do not screw or unscrew Dental Tech abutment screws for impressions with unsuitable equipment.
- Use them for the above-mentioned purpose only.
- Do not modify its original structure either of the implant impression or its abutment screw.
- Dental Tech implant impressions cannot be left permanently in the oral cavity.

PACKAGING

- Dental Tech tools are supplied as non sterile in heat-sealed pouches unsuitable for autoclave sterilisation.

CLEANING, DISINFECTION AND STERILIZATION

- Dental Tech tools can be used again and therefore it has to be washed, disinfected and sterilised prior to their usage.
- Cleaning, disinfection and sterilization operations must be carried out by qualified personnel under the direct responsibility of the doctor; sterilization must be carried out through the use of regularly calibrated machinery and in perfect state.
- Wear disposable gloves during all cleaning, disinfection and sterilization operations.
- Dental Tech has validated the manual cleaning and disinfection method below. Do not use automated cleaning and disinfection methods because they are not validated.

PREPARATION TO THE POINT OF USE

- Set aside damaged or beveled tools.
- Proceed with manual cleaning without letting the organic material dry on the instruments.

MANUAL CLEANING

- Just after the use of Dental Tech equipment, place the equipment into ultrasound bath with a peracetic acid based solution at concentration of 2% (NO GLUTAR- ALDEHYDE OR SODIUM HYPOCHLORITE), as long as 18 minutes.
- Afterward rinse carefully with drinking water.

MANUAL DISINFECTION

- Place the equipment into a container with a peracetic acid based solution at concentration of 4% (NO GLUTARALDEHYDE OR SODIUM HYPOCHLORITE), as long as 15 minutes.
- Rinse generously with drinking water.
- Examine the equipment and make sure there are no organic remains. Carefully scrub the outer parts with a non-metal rigid bristled brush.

MANUAL RINSE

- Place the equipment into ultrasound bath with distilled water, and wash it for approx. 18 minute and then rinse carefully with distilled water.

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