## Italiano

**I**

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**GB**

### GENERAL WARNING

- Dental Tech tap surgical wrenches shall be used by specialised medical staff and dental technicians only.
- The use of tap surgical wrenches not belonging to the endosseous system manufactured by Dental Tech, may impair its functionality.
- The product is manufactured and applied out of our control and responsibility. The manufacturing company cannot be held responsible for any deriving damage.
- Prior to using any Dental Tech product inspect the packaging and device for damages.
- Tap surgical wrenches are cutting tools. Take precautions when washing and cleansing them especially if they have been contaminated by recent use. Wear protection goggles, surgical mask and disposable gloves.
- At the end of its useful life the medical device must be disposed of in accordance with national laws regarding waste disposal.
- Dental Tech cannot be held responsible for any damage resulting from inobservance of this warning.

### INTENDED USE

Tapper surgical wrench for dental use.

### INDICATIONS FOR USE

- In order to set a surgical site to insert dental implants, we consider Dental Tech tap surgical wrenches as the most suitable tools to tap bone, to create the appropriate thread to insert the implant in the site previously created by the drill.
- Target population: adults suffering from edontulia.

### METHOD OF USE

- Sprinkle tap surgical wrenches generously and constantly with sterile physiological solution previously cooled down.
- Tap surgical wrenches are manually used through special wrenches or together with a surgical micro-motor, thanks to its special knob and relevant tap adapter, with constant sterile saline irrigation.

### CONTRAINDICATIONS

- When using Dental Tech tap surgical wrenches never lever or bend them.
- Use them for the above-mentioned purpose only.
- Do not wash drills and tap surgical wrenches in the same bath, but carefully separate.
- To prevent tools from deteriorating and bones from overheating, **do not use tap surgical wrenches for more than 20 repetitions.**
- Do not change its original structure.
- Do not exceed **20 RPM** when thread tapping with counterangle.

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### PACKAGING

- Dental Tech tools are supplied as non sterile in heat-sealed pouches in containing the leaflet.
- Pouches unsuitable for autoclave sterilisation.

### CLEANING, DISINFECTION AND STERILIZATION

- Dental Tech tools can be used again and therefore it has to be washed, disinfected and sterilised prior to their usage.
- Cleaning, disinfection and sterilization operations must be carried out by qualified personnel under the direct responsibility of the doctor; sterilization must be carried out through the use of regularly calibrated machinery and in perfect state.
- Wear disposable gloves during all cleaning, disinfection and sterilization operations.

Dental Tech has validated the manual cleaning and disinfection method below. **Do not use automated cleaning and disinfection methods because they are not validated.**

### PREPARATION TO THE POINT OF USE

- Disassemble the tapper from the handpiece immediately after use.
- Set aside damaged or beveled tools.
- Proceed with manual cleaning without letting the organic material dry on the instruments.

### MANUAL CLEANING

- Just after the use of Dental Tech equipment, place the equipment into ultrasound bath with a peracetic acid based solution at concentration of 2% (NO GLUTAR-ALDEHYDE OR SODIUM HYPOCHLORITE), as long as 18 minutes.
- Afterward rinse carefully with drinking water.

### MANUAL DISINFECTION

- Place the equipment into a container with a peracetic acid based solution at concentration of 4% (NO GLUTARALDEHYDE OR SODIUM HYPOCHLORITE), as long as 15 minutes.
- Rinse generously with drinking water.
- Examine the equipment and make sure there are no organic remains. Carefully scrub the outer parts with a non-metal rigid bristled brush.

### MANUAL RINSE

- Place the equipment into ultrasound bath with distilled water, and wash it for approx. 18 minute and then rinse carefully with distilled water.

### DRY

- Dry perfectly in aseptic conditions (gauze, paper, filtered air...) and store the instruments individually bagged using Medical Paper C and FILM PET/PP Peel (ISO Reference 11607-1).

### STERILIZATION (according to ISO 17665-1)

- Dental Tech validated the following Autoclave moist heat sterilization cycle:
- Pre-vacuum time according to autoclave builder specifications.
  - Temperature: 134 °C.
  - Holding time: 3 minutes.
  - Drying time: minimum 10 minutes.

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